



Image Access for Universal Medical Imaging Referrers



Visage Referrer Gateway Access

Utilising the **Visage 7** Thin Client technology, the Visage Referrer Gateway and Visage Ease App for iPad iPhone, and iPod-Touch, you are provided with immediate server-based access to DICOM images without cumbersome image transfer. Compared to printed film or patient CDs, the Visage 7 solution provides a much richer and more efficient means of communication.:

1. For your initial login, download and install the native Visage thin client viewer by visiting:

<http://images.umic.com.au/download>

2. Select the Client that suits your operating system (Mac/Windows)



2. Login using the following credentials:


Server Name: images.umic.com.au

And the username and password supplied by your UMIC Administrator



Search for your Patients

3. Specify a date range and optionally enter additional search criteria such as patient name, birth date, study date.

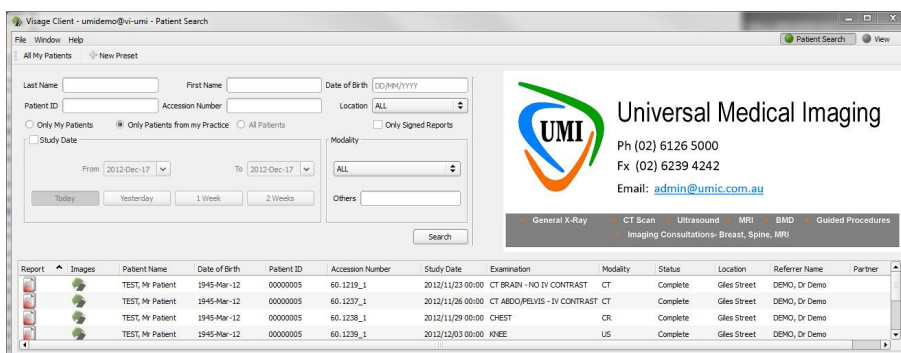

4. Double click on the icon in the Report or Image column respectively to see the report or the images. Double click the whole line to view both.

